

Glove Donning Guide for Biogel® Indicator® Systems

CLOSED Glove Donning with the cuff over the hand

Don the sterile gown and open the glove pack and inner wraps¹ as recommended.



1. Do not slip hands through the gown cuff. The cuffs should be loose as shown in the picture.



2. The glove pack is opened by the circulating nurse. Grab the inner wraps with the fabric of the gown, ensuring the hands are kept inside the gown sleeves.



3. Place the inner wraps in the sterile zone and ensure the fingers are pointing towards you.



4. Open the inner wrap (the coloured gloves), ensuring the hands are kept inside the gown sleeves at all times.



5. Pick up the first glove with your dominant hand (the right glove with your right hand or the left glove with your left hand).



6. Gently flip the glove over, so the glove now rests on your right or left forearm. The glove thumb is now lined up with your thumb.



7. Using your other hand which is still inside the gown sleeve ...



8. ...pull the edge of the glove over the back of your hand.



9. Wriggle fingers and thumb into place as gown and glove slide together up your arm.



10. The glove is now fully on. Ensure that the donned glove always completely covers the woven cuff.



11. Pick up the other glove with your ungloved hand, flipping the glove over onto your forearm and glove this hand in the same manner.




12. Using your gloved hand, pull the edge of the glove over the back of your hand.



13. Wriggle fingers and thumb into place, pull glove up your arm until it is fully donned.



14. Don the outer gloves.

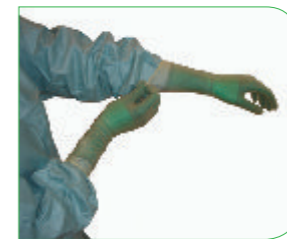

15. Ensure that the outer gloves are pulled well over the inner gloves (approximately 2.5cm) to avoid roll or slip down.

Changing a contaminated/punctured glove during the surgical procedure



16. With the Biogel® Puncture Indication Systems, if a glove puncture occurs in the presence of fluid, a dark coloured patch appears through the outer glove at the site of perforation.



17. To change a contaminated glove during the procedure, remove the outer glove by pulling the cuff over the palm, so that the glove turns inside out. Dispose of the contaminated/punctured glove.



18. Check the inner glove for punctures. If punctured, remove and replace as per local recommendations. Otherwise, clean and dry with a sterile wipe.

Open a new glove package of the correct Biogel Indication System works in the intended way. Don the new outer glove, ensuring that the cuff of the outer glove is pulled well over the cuff of the inner glove by 2.5 cm to avoid roll or slip down.

For video instructions on how to don gloves and surgical gowns please see the Mölnlycke web site.

References: ¹Folin A. Opening methods for the indicator double gloving system packaging. Safe Hands 2/2008. Mölnlycke Health Care AB.



Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, SE-402 52 Göteborg, Sweden. T +46 31 722 30 00. F +46 31 722 34 00. www.molnlycke.com
The Mölnlycke Health Care and Biogel names and logos are registered trademarks globally to the Mölnlycke Health Care Group of companies. Copyright (2012)

